
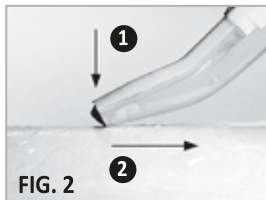
Cortical Bone Collector

CE
0123

SAFESCRAPER TWIST

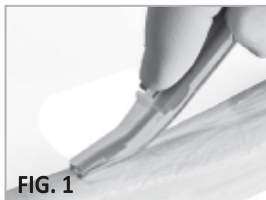- IT
- EN
- DE
- FR
- ES
- PT
- DA
- NL
- SV
- NO
- LT
- LV
- PL
- RO
- HU
- BG
- CS
- TR
- EL
- RU
- HR
- SL
- AR
- ZH
- UA
- KO
- JA

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Device used to draw and collect autologous bone in regenerative applications in oral surgery.

- **Before use ensure that the packaging is intact.**
- **The device must exclusively be used by qualified medical staff.**
- **Always use sterile gloves when handling the device, taking care not to cut them with the sharp edges of the blade.**
- **Discard after use in special containers for medical waste in conformity with the decrees in force.**
- **META is not liable for the incorrect use of the product.**

- 1 Open the package on the side with the symbol " ". Lay the instrument on the sterile field always handling it in maximum sterile conditions.
- 2 The device must be gripped with the transparent tank always facing the bone and in such a way as to guarantee accurate movements during use (see figures 1).
- 3 The bone is collected by pushing the end of the device towards the bone surface and simultaneously pulling the device itself backward. (see figures 1). The bone can also be collected with slanting and sideways movements.
By varying the angle of the instrument and the pressure exercised, the desired consistency of the bone lamellae or plates can be obtained. The Safescraper® straight tank may contain up to 3 cc. of bone, whereas the Safescraper®curve tank may contain up to 2.5 cc.

- 4 To collect the bone, keep the transparent tank facing downward, press the lock button sideways and slide the tank clockwise (see figure 3-4).

To continue the bone collection procedure or before disposing of the Safescraper®, put the tank back in its original position and keep the button pressed laterally to trigger the tank lock position (see figure 5).

To transfer the bone lamellae or plates to the receiving site, any type of forceps or spoon can be used (see figure 6).

Suggestions:

In the event of bad cutting quality, change the angle of the instrument and the pressure exercised on the bone.

If no improvement is obtained, the blade might be worn and the device must therefore be replaced.

- **The device may be used once only. It is absolutely impossible to clean and/or resterilize it after use.**
- **Caution: if used with another patient may lose performance and functional features and cause cross-contamination**

Warnings:

The surgeon must establish whether the patient is suited to bone graft.

The surgeon must adopt all the necessary precautions to prevent the risks of infections and complications associated with bone graft procedures.

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